

Electronic Reading Aid




User's Guide

Open box and unpack contents

- 1 x Electronic Reading Aid (ERA) with power/video cable
- 1 x power adaptor
- 1 x carry bag
- 1 x user's guide

A quick look at the Electronic Reading Aid (ERA)

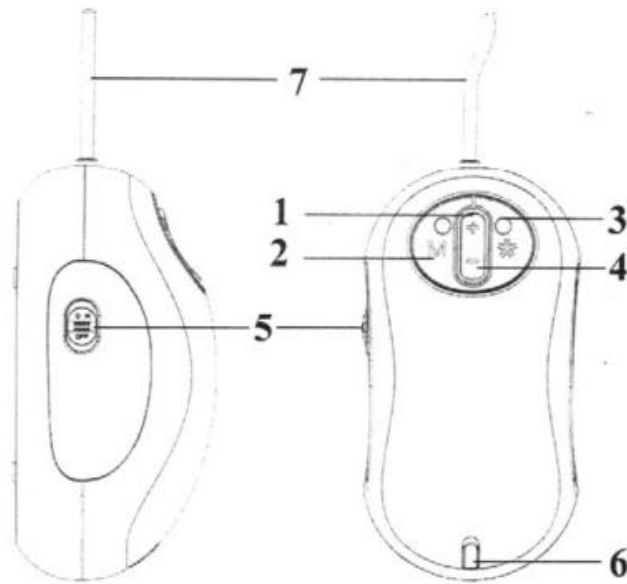

1. Magnify
2. Mode
3. Freeze & D-freeze
4. De-magnify
5. Power on/off
6. Power indicator
7. Y-shaped power/video cable

How to use the ERA?

1. Insert the small end of the power adaptor into the black end of the Y-shaped power/video cable from the Reading Aid.
2. Plug the power adaptor into a wall socket.
3. Insert the yellow RCA plug on the video cable into one of your television's yellow video input connectors. Refer to your television owner's manual for more information locating these connectors.
4. Turn on the television. If necessary, switch your television to the appropriate input channel.
5. Turn on the ERA by sliding the on/off button to ON position. Then the power indicator will light.
6. Put the ERA on newspapers, magazine etc and then you can read their big images from TV. Now you can enjoy your reading.

7. Key definitions:

- 1) M: Switch reading mode among Full color, Black on White, White on Black, Yellow on Black, Blue on Yellow, Blue on White, Yellow on Bl
- 2) + : Magnify
- 3) - : De-magnify
- 4) *: Freeze & D-freeze

Safety:

- The ERA is not waterproof, so keep it dry.
- Use the ERA only at -5°C-50°C.
- The white LEDs which illuminate the ERA target field are very bright. Do not stare directly into these LEDs as it may damage your eyes.
- Use only the power adaptor that came with the ERA. Other adaptors may damage the unit.

Warranty Card

Your Reading Aid has a one year guarantee against any manufacturing defect!

Date of purchase:		Product No.	
Seller's info	Name:		
	Address:		
	Tel:		
Defect description			
Cause			
User's info	Name:		
	Address:		
	Tel:		

Distributed by:***Mattingly Low Vision, Inc.***

111 Rio Flor Place

El Paso, TX 79912

www.mattinglylowvision.com

888.642.0842

